Specifications			
	Blade	Incision Depth	Incision Length
tenderfoot micro-preemie	surgical steel	0.65 mm	1.40 mm
tenderfoot preemie	surgical steel	0.85 mm	1.75 mm
tenderfoot	surgical steel	1.00 mm	2.50 mm
tenderfoot toddler	surgical steel	2.00 mm	3.00 mm

For Professional Use

! If package containing device is damaged or open, do not use. Used devices are potentially infectious. Handle with appropriate care and dispose of used devices in accordance with standard methods of biohazard control.

Materials

Materials Required But Not Provided

- Antiseptic swabs, 70% isopropanol
- Sterile gauze pads, 2" x 2"
- Blood collection container, capillary tube, filter paper

Summary of Procedure

The so-called safe area to inflict a neonatal or infant heel wound is, "Marked by a line extending posteriorly from a point between the 4th and 5th toes and running parallel to the lateral aspect of the heel, and a line heel wound is "Marked by a line extending posteriorly from a point between the 4th and 5th toes and running extending posteriorly from the middle of the great toe parallel to the lateral aspect of the heel and a line running parallel to the medial aspect of the heel". * Illustrated as follows:

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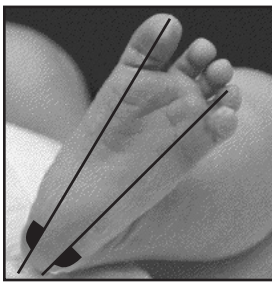
*Blumenfeld TA, Turi GK, and Blanc WA: Recommended Site and Depth of Newborn Heel Skin Punctures Based on Anatomical Measurements and Histopathology. Lancet, 18110:230-233, 1979.

With the baby in a supine position, the heel incision site is cleaned with an antiseptic swab and air dried. The tenderfoot device is properly positioned on the heel and triggered. The first drop of blood is wiped away using a dry sterile gauze pad. The blood specimen is then collected in the desired capillary tube or container. Following blood collection, gently press a sterile pad to the incision site until bleeding has ceased. The baby's heel must then be monitored for late bleeding and inflammation.

Detailed Procedure

Note
It is important not to handle sick babies unless it is absolutely necessary.

1. The ideal posture for this procedure is with the baby in a supine position with the knee at the open end of a bassinnet. This position allows for the foot to hang lower than the torso, improving blood flow. When the baby is in an acceptable position for this procedure, clean the incision area of the heel with an antiseptic swab. Allow the heel to air dry. Do not touch the incision site or allow the heel to come into contact with any non-sterile item or surface. (See Figure 1)
2. Remove the appropriate tenderfoot device from its blister pack taking care not to rest the blade slot end on any non-sterile surface. (See Figure 2)
3. Remove the safety clip. Note: The safety clip may be

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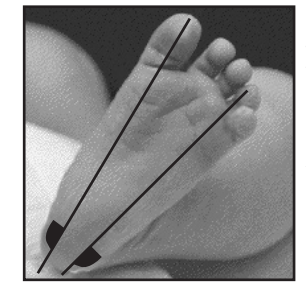
replaced if the test is momentarily delayed; however, prolonged exposure of any tenderfoot device to uncontrolled environmental conditions prior to use may affect its sterility. Once the safety clip is removed, DO NOT push the trigger of touch the blade slot. (See Figure 3)

4. Raise the foot above the baby's heart level and carefully select a safe incision site (avoid any edematous area or site within 2.00 mm of a prior wound). Place the blade-slot surface of the device flush against the heel so that its center point is vertically aligned with the desired incision site. (See Figure 4)
5. Ensure that both ends of the device have made light contact with the skin, and depress the trigger. After triggering, immediately remove the device from the infant's heel. (See Figure 5)
6. Using only a dry sterile gauze pad, gently wipe away the first droplet of blood that appears at the incision site. (See Figure 6)
7. Taking care not to make direct contact with the collection container or capillary tube, fill to the desired specimen volume. (See Figure 7)
8. Following blood collection, gently press a dry sterile gauze pad to the incision site until bleeding has ceased. This step will help prevent a hematoma from forming. (See Figure 8)
9. Whether or not to bandage the baby is a controversial issue because of skin sensitivity and potential bandage aspiration. However, the incision should be noted by the primary care nurse to ensure that the heel can be monitored for bleeding and inflammation.

Conditions Affecting the Procedure

- Heel edema
- Re-incision of prior wound site
- Inflamed heel
- Excessive pressure and skin indentation from placing the device on the heel, resulting in deep and hazardous wound depths

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Limitations of the Procedure

- Care and proper procedure must always be followed to avoid injury
- Poor vascularization may cause inadequate blood flow

Note
Warming the heel to 42 to 44 C will improve blood flow, but temperatures above 44 C will burn the heel.

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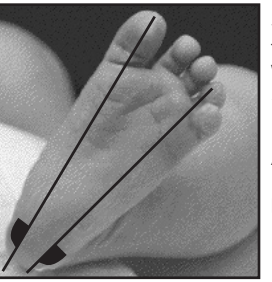
Tenderfoot® is a registered trademark of Accriva Diagnostics, Inc.
Patent No. 5,314,441; 5,772, 677; 6,010,519

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