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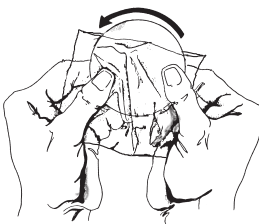
Fig. 4

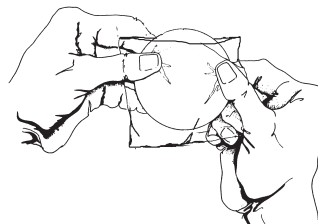
Fig. 5

International Technidyne Corporation  
8 Olsen Avenue, Edison, NJ 08820 USA  
T: 800.631.5945 • F: 732.248.1928  
www.itcmed.com

a subsidiary of THORATEC® Corporation

Surgicutt® is a registered trademark of ITC.  
Trademark# 1,460,731

# Surgicutt®

## Bleeding Time Device

P A C K A G E I N S E R T

English / Deutsch / Français  
Italiano / Español

## Product Description

Surgicutt® Bleeding Time Blotting Paper is a unique product designed to safely and effectively absorb blood from a bleeding time wound.

Highly absorbent blotting paper is encased in a protective shield that safeguards against direct contact with patient blood. The shield is transparent so that the user can see if excessive blood is absorbed or if additional paper is required.

Because the paper fibers are extremely absorbent, blood droplets are rapidly wicked away from the incision site. This enhances visibility and minimizes the potential for contact between paper and wound, an error in bleeding time testing.

## Specifications

- Highly absorbent
- Safe
- Easy to use
- 9 cm diameter
- Approximately 365% more absorbent than #1 filter paper

## Product Use

(Refer to the Surgicutt Bleeding Time device package insert for detailed test procedure.)

1. Prepare the patient and perform the incision according to accepted bleeding time test protocol.
2. To use Surgicutt Bleeding Time Blotting Paper open the protective shield by tearing at the perforated strip. (Fig. 1)
3. Push paper through the opening in the protective shield. (Fig. 2)
4. Every 30 seconds use blotting paper to wick the flow of blood by touching paper to the drop of blood. Do not touch paper directly to the incision, so as not to disturb formation of the platelet plug.
5. Rotate paper to expose a clean area of paper (Figs. 3, 4 and 5)
6. At the next 30-second interval, blot using the newly exposed portion of paper.
7. Continue to wick blood at 30-second intervals until the test is completed.

After use, blotting paper should be considered contaminated and handled according to institutional procedures regarding the disposal of contaminated materials.

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