## Platelet Reactivity Test

### VerifyNow Assay WQC

**Package Insert**  
**English**

Technical Support  
(US) 800-579-2255  
(Outside US) 858-263-2502

### Overview

The VerifyNow Assay Wet Quality Control (WQC) is intended to be used with the VerifyNow Instrument and the VerifyNow Test Device as a basis of quantitative quality control. Additional information about running this control is located in the Package Insert of the VerifyNow Test kit.

#### Reagents

- VerifyNow Level 1/Level 2 WQC diluent tubes containing 2 mL of optically absorbent suspension with bovine serum albumin and a preservative.
- VerifyNow Level 2 WQC pellets containing a peptide in buffered solution and colored pink.

Each VerifyNow Assay WQC Kit contains material for six Level 1 or Level 2 controls.

### Precautions

For *in vitro* diagnostic use

Used WQC material in the VerifyNow Test should be discarded in accordance with institutional guidelines for biohazardous waste disposal.

### Level 1

Gently invert the diluent 4 to 5 times to mix the reagents. VerifyNow Level 1/Level 2 WQC Diluent is now ready for use as a Level 1 Control.

### Level 2

Immediately before use, open the vial containing the VerifyNow Level 2 WQC pellet. Remove the stopper from the VerifyNow Level 1/Level 2 WQC Diluent tube by twisting and pulling the cap simultaneously. Hold the tube vertically and insert the pellet into the diluent. Replace the stopper on the tube by pressing and turning simultaneously. Gently invert 4 to 5 times to mix the reagents. VerifyNow Level 2 WQC is now ready for use.

See VerifyNow User Manual for additional instructions.

#### Storage and Stability

Control material should be stored at 15° to 25°C (59° to 77°F). Control material is suitable for use until the expiration date stamped on the label. Control material should be used within 15 minutes of reconstitution. Do not freeze.

### Limitations

Control material should be used prior to the expiration date on the label. If the control material does not produce a result within the range stated on the test device pouch, repeat the procedure using a new assay device and new VerifyNow Assay WQC material. If the result is still out of range, call Technical Support for further assistance.

#### Expected Values

#### VerifyNow Aspirin Test

See device pouch for expected values.

#### VerifyNow Ilb/IlIa Test

See device pouch for expected values.

#### VerifyNow P2Y12 Test

See device pouch for expected values.

### VerifyNow PRUTest

See device pouch for expected values.

<b>LOT</b>	Lot Number
<b>REF</b>	Catalog Number
	Manufacturer
<b>EC/REP</b>	Authorized Representative in the European Community
<b>IVD</b>	In Vitro Diagnostic Use
	Use By
	Temperature Limits
<b>CONTROL L1</b>	Control Level 1
<b>CONTROL L2</b>	Control Level 2 Diluent
	Do not freeze
	Consult Operating Instructions
	Caution, consult accompanying documents.
	Contains sufficient for < n > tests
<b>RX ONLY</b>	CAUTION: US Federal Law restricts this device to prescription only
	Biological Risks
	Do not reuse

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Accriva Diagnostics, Inc.
MDSS GmbH